



Multiplexed POC System for HIV and Co-Infection Serodiagnosis

**Michael Lochhead, PhD
mBio Diagnostics, Inc.**

**HIV Diagnostics Conference
March 24, 2010
Orlando, FL**

Background

- mBio Diagnostics, Inc., Boulder CO
- Collaboration with UCSD
 - Drs. Schooley and Benson
- Point-of-care (POC); low-cost, instrumented tests
 - Global health focus



- **MAJOR THEMES**
 - Multiplexed assay panels at point of care
 - Rapid test workflow & cost with lab quality output




mBio SnapEsi™ System


- Fluorescence immunoassays
- Digital and quantitative
- Designing for CLIA-waiver

Address limitations of visually read rapid tests

- Interpretation
- Training; QA/QC
- Connectivity

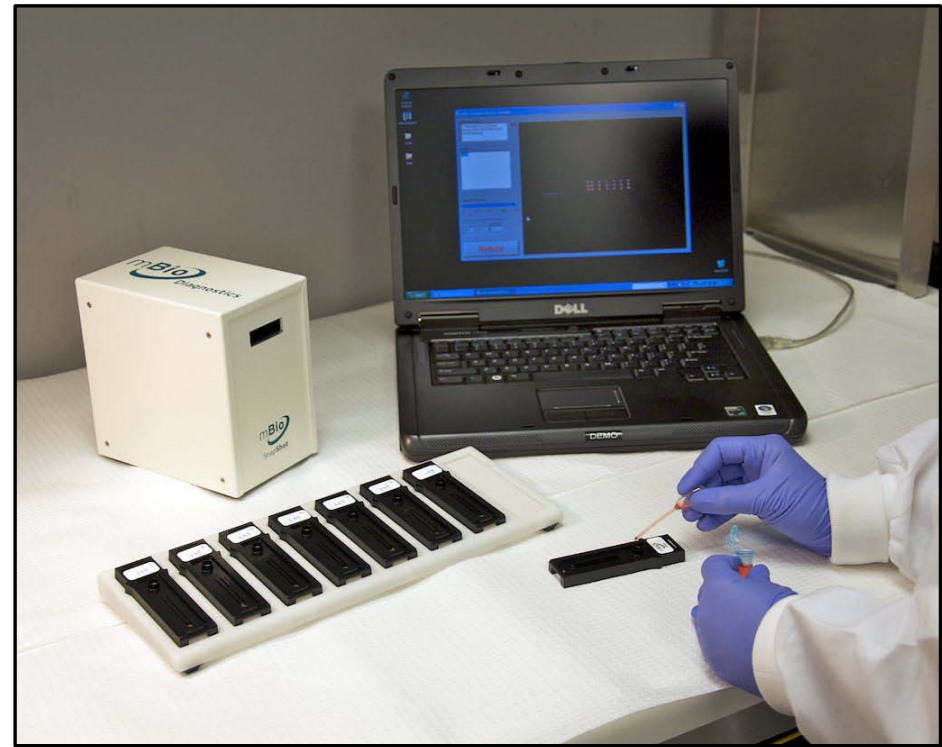


Multiplexed Serology Assay

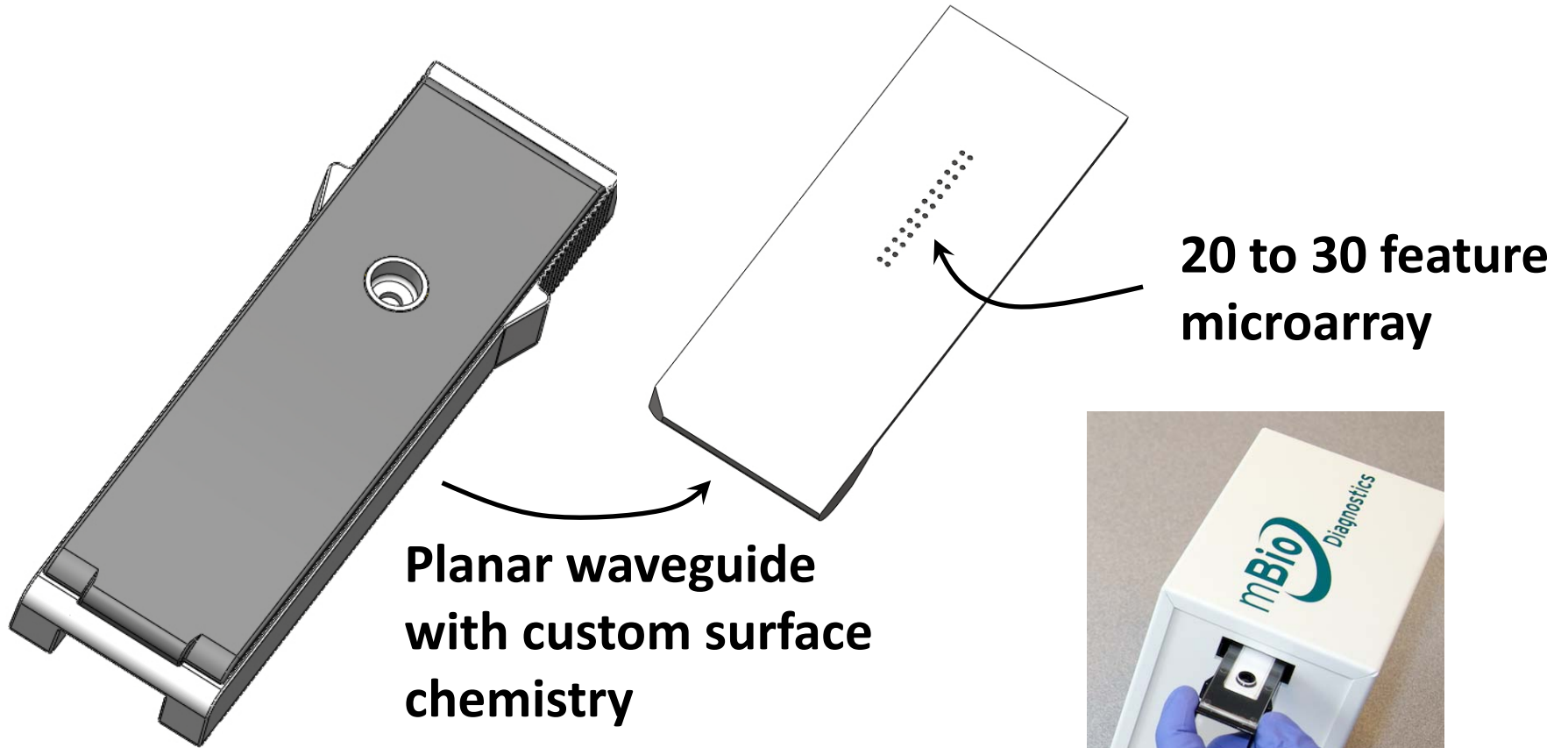


Multiplexed Serology Assay

- **Low density microarray in fluidic channel**
 - Printed antigen spots
 - Sample and procedural controls
- **Detect antibodies in serum, plasma, blood**

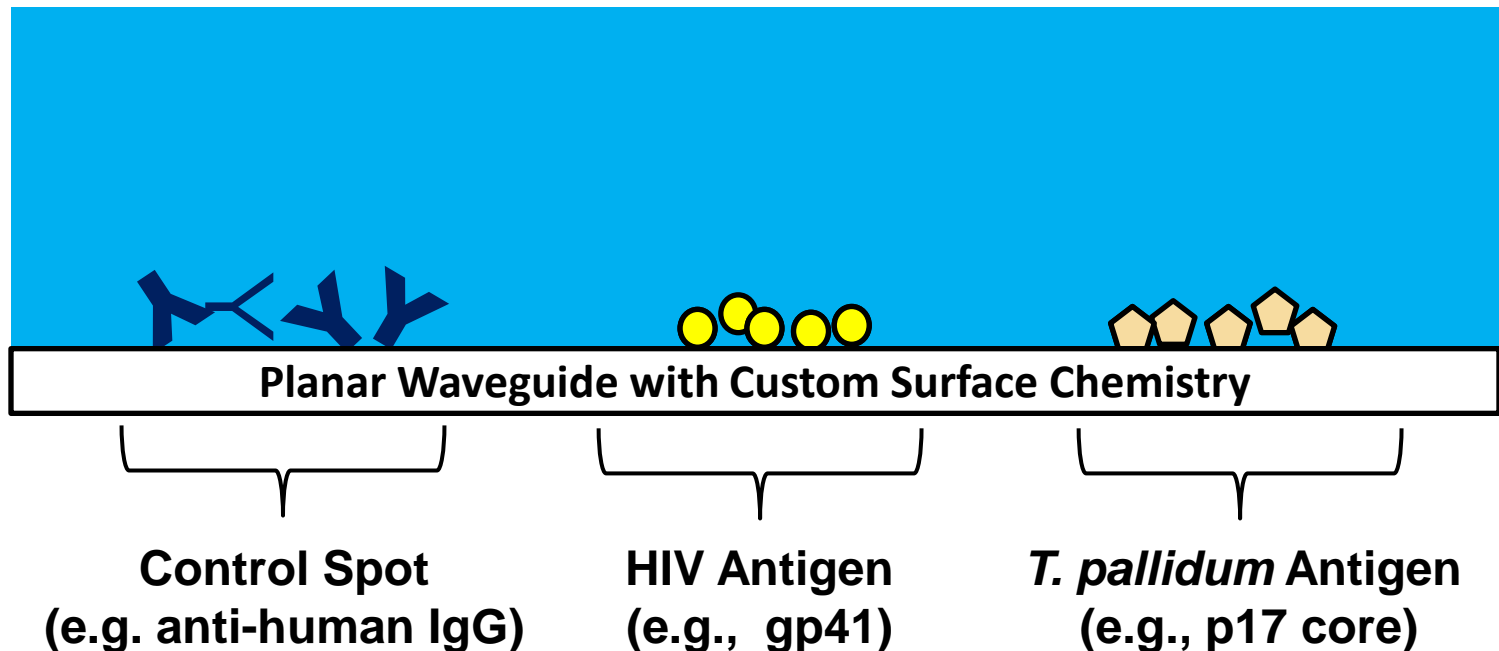


Multiplexed Serology Assay

- **Low density microarray in fluidic channel**
 - Printed antigen spots
 - Sample and procedural controls
- **Detect antibodies in serum, plasma, blood**

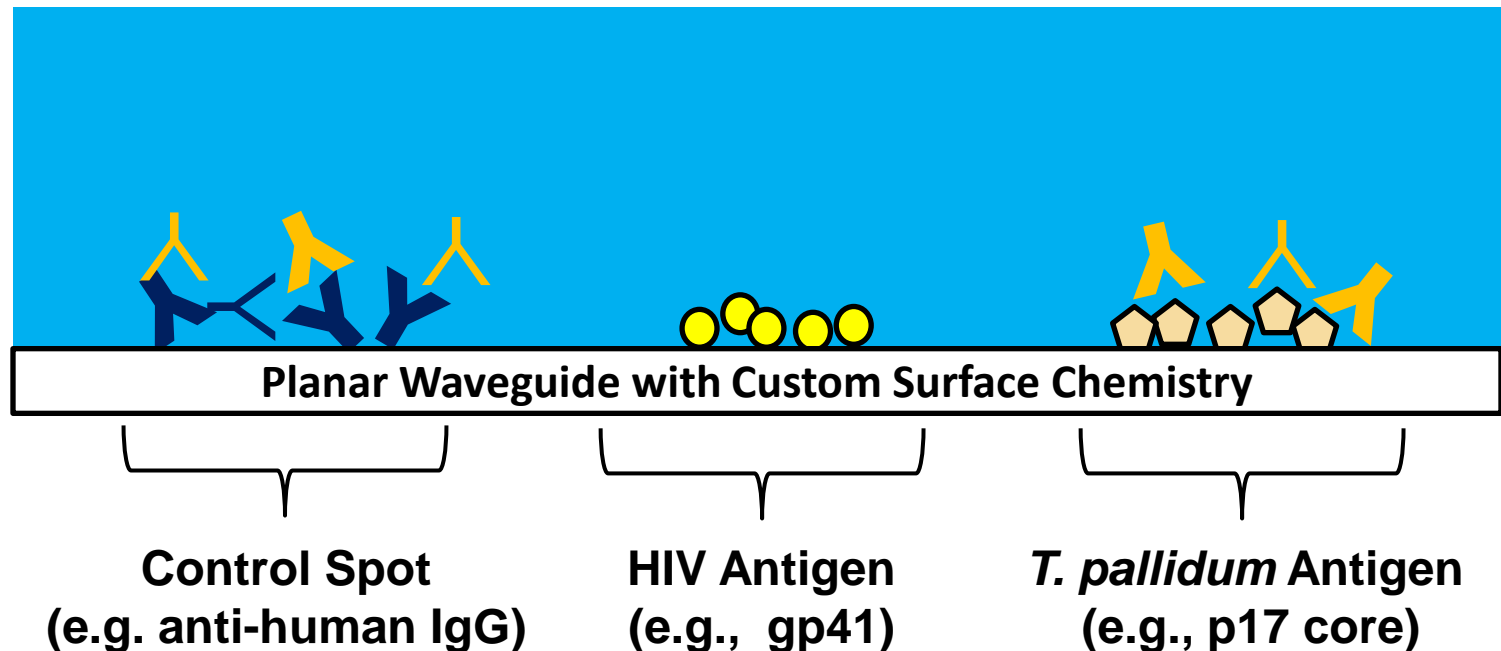


Multiplexed Serology Assay

- **Low density microarray in fluidic channel**
 - Printed antigen spots
 - Sample and procedural controls
- **Detect antibodies in serum, plasma, blood**

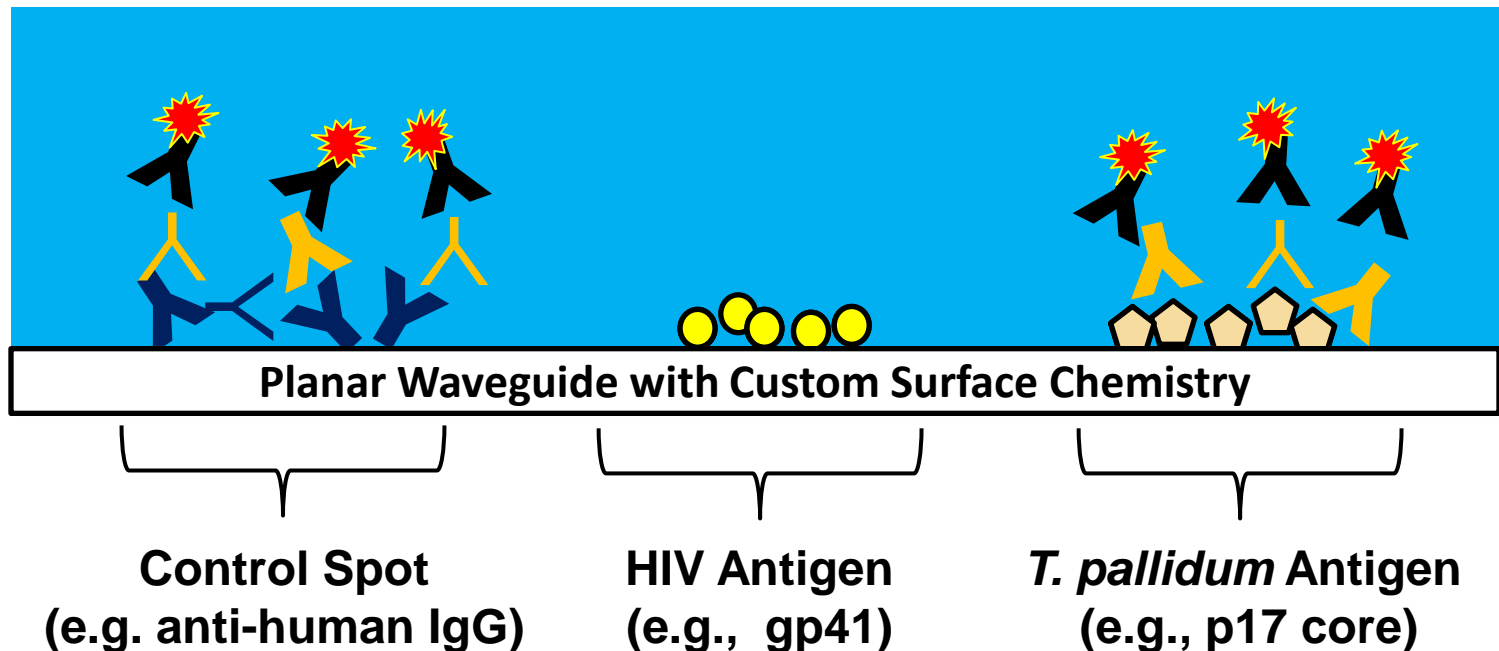


Multiplexed Serology Assay

- Fluorescence imaging detection
- Patent-protected planar waveguide design
- Robust, reproducible illumination at very low cost

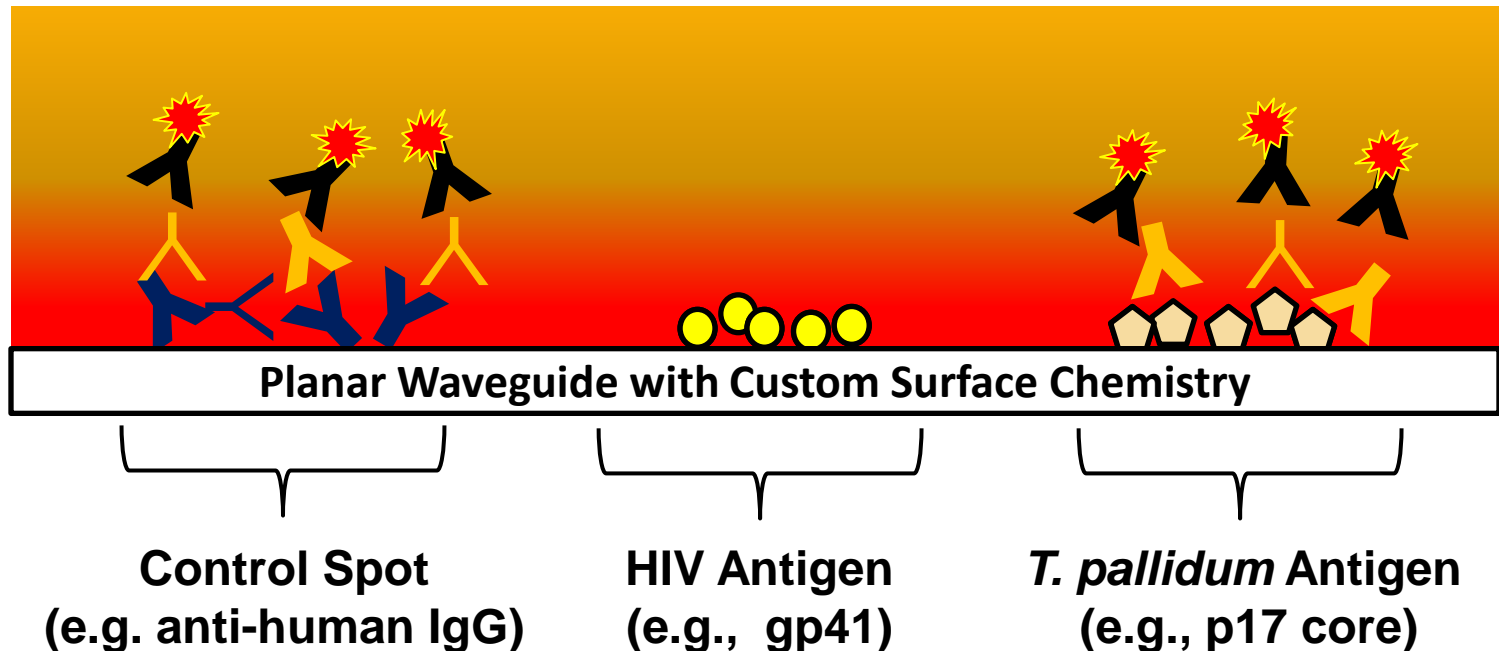


mBio SnapEsi™ System

Low Cost Fluorescence Imaging

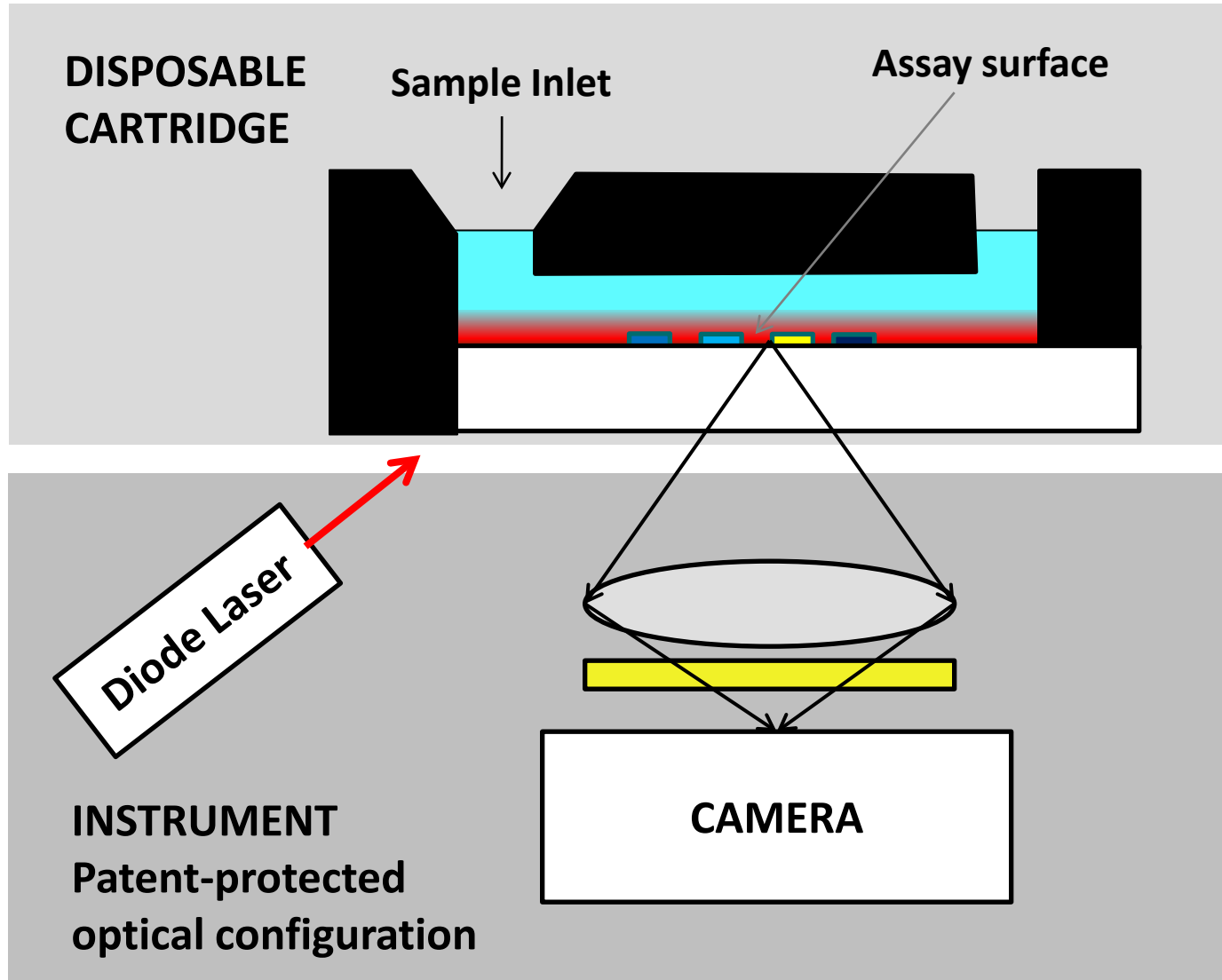




SnapEsi™ HIV-1/Syphilis Workflow

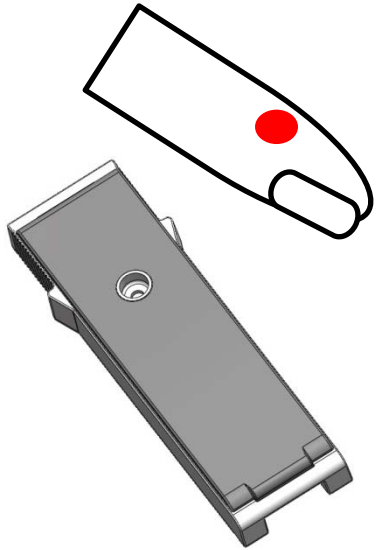


- **5 to 10 microliter sample**
- **Total assay time ~25 min**
- **Batch processing; one operator:**
 - 16 cartridges in parallel
 - 60 to 80 per shift
- **Read window: 10 min to hours**


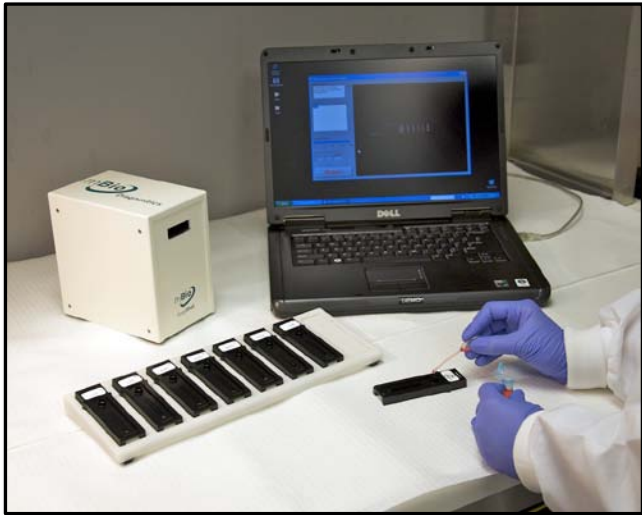





SnapEsi™ Ab Reactivity Output

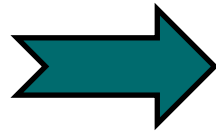


Pathogen	Status	Antibody Reactivity	Signal / Cutoff
HIV	POSITIVE	gp41	5.0
		gp120	5.4
		p24	7.8
<i>T. pallidum</i>	NEGATIVE	p47	0.0
		p17	0.0
		TmpA	0.0
Control	Good Test	Anti-Hu IgG	4.1

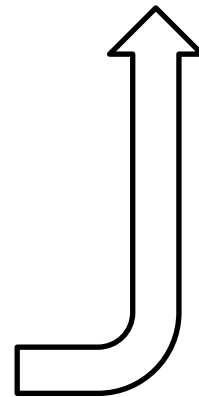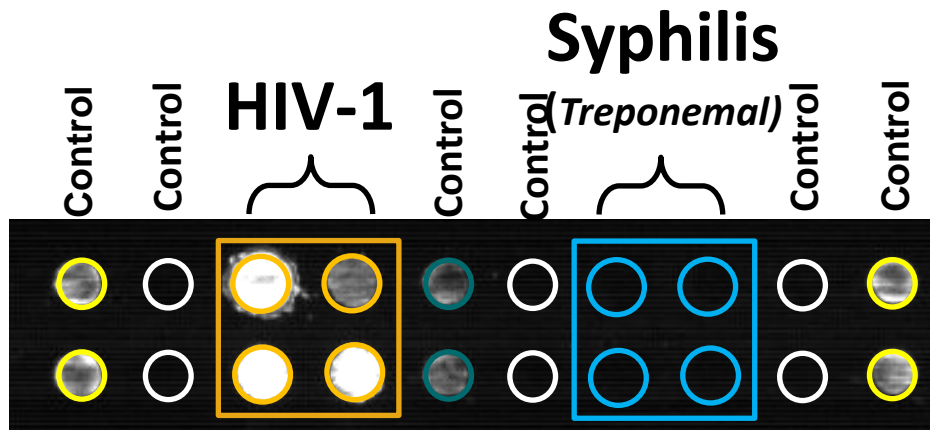

HIV-1 / Syphilis Combination Assay

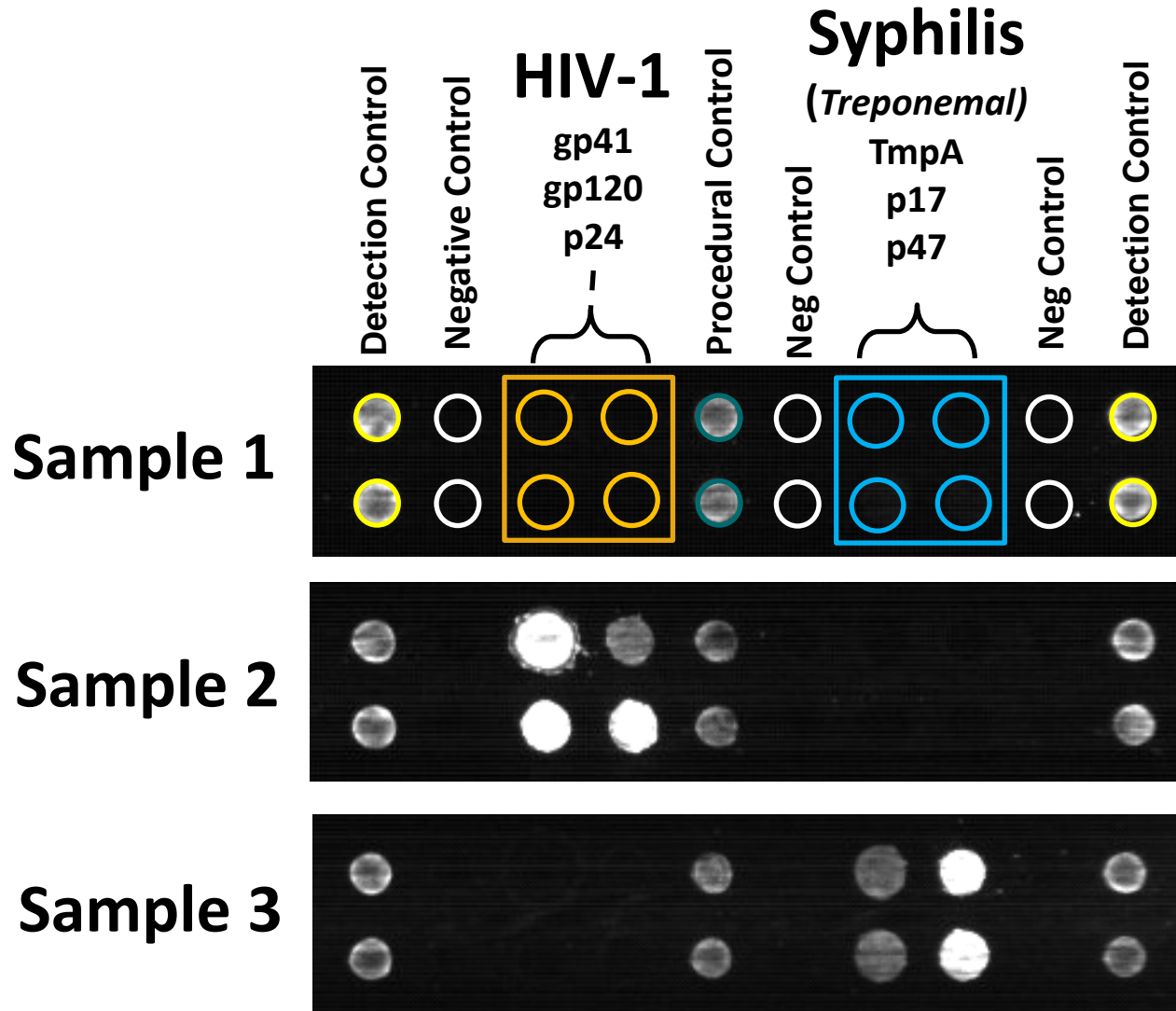



HIV-1 / Syphilis Combination Assay

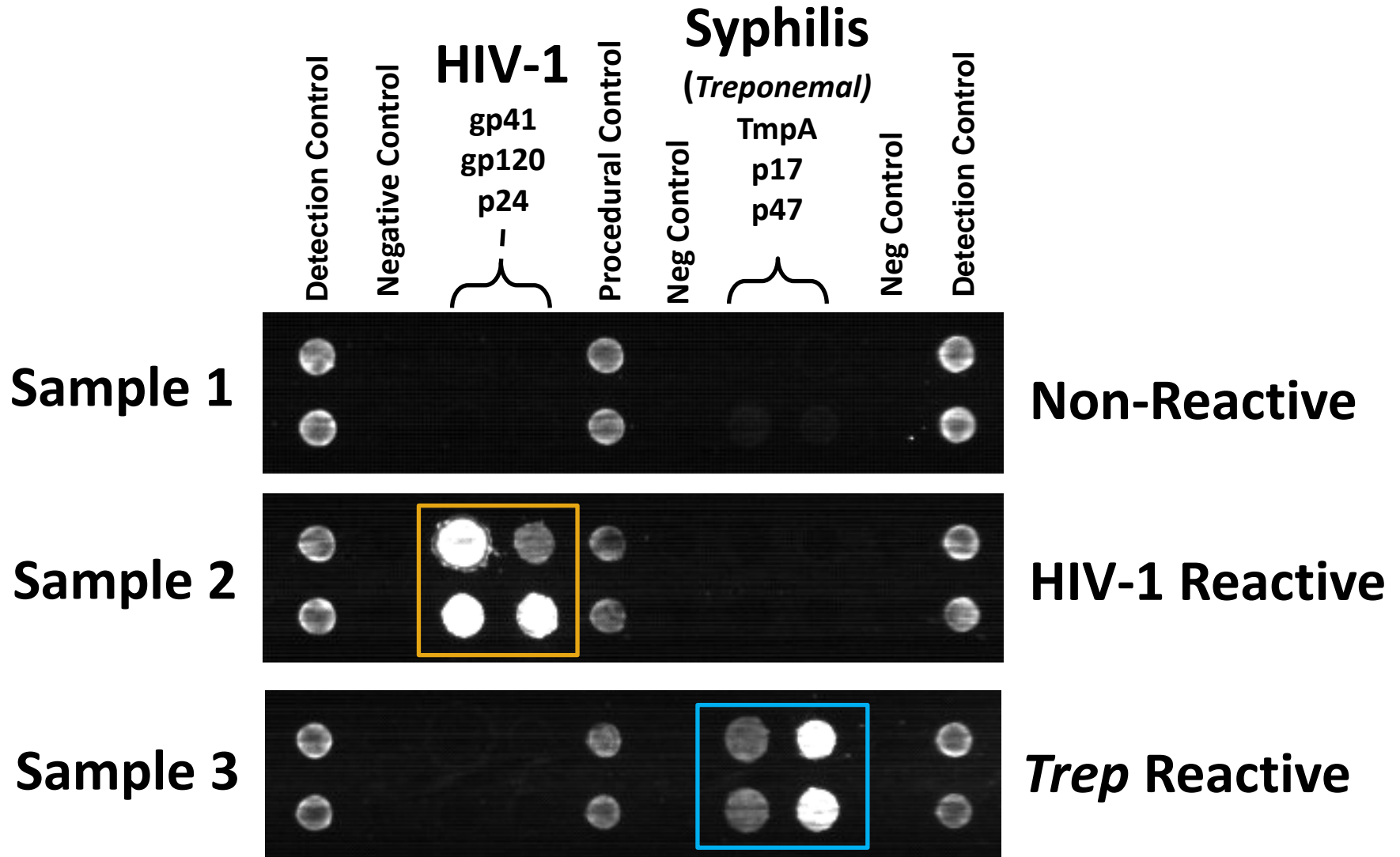



Example Data Output

Treponemal Ab Reactive Sample

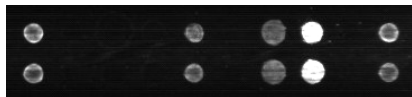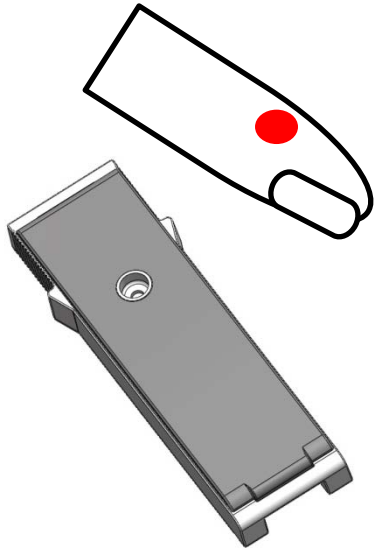

Pathogen	Status	Antibody Reactivity	Signal / Cutoff
HIV	NEGATIVE	gp41	0.0
		gp120	0.1
		p24	0.0
<i>T. pallidum</i>	POSITIVE	p47	5.7
		p17	3.7
		TmpA	8.8
Control	Good Test	Anti-Hu IgG	6.1



HIV-1/Syphilis Clinical Sample Results

- Serum Samples from University of California San Diego Clinical Micro Lab (Siemens ADVIA Centaur analyzer)
- Commercial controls (including negatives)
- ❖ Sensitivity and Specificity based on same dataset used to establish cutoffs

HIV		n = 170	
True Pos	False Neg*	True Neg	False Pos
54	1	115	0
Sensitivity		Specificity	
98.2%		100.0%	
*Also negative on OraQuick®			



HIV-1 Seroconversion Panel

PATH (Seattle, WA) Evaluation Result

		mBio SnapEsi™	Abbott EIA	Maxim Western Blot	Siemens Advia Centaur
Sample	Days since 1st bleed	Result	s/co	Result	s/co
PRB956-01	0	NEG	0.1	NEG	0.1
PRB965-02	5	NEG	0.1	NEG	0.1
PRB965-03	7	NEG	0.2	NEG	0.7
PRB965-04	12	POS	>17.7	IND	>50
PRB965-05	14	POS	>17.7	POS	>50
PRB965-06	21	POS	7.6	POS	>50
		PATH Result	Package Insert Data		



mBio SnapEsi Syphilis (*Treponemal*) Clinical Serum Sample Results

***T. pallidum* Positives**

- **46 confirmed *Trep*-positive (TPPA)**
- **45 SnapEsi POS; one indeterminate**
- **97.8% Sensitivity**


RPR Negatives

- **111 RPR negatives**
- **108 SnapEsi *Trep* NEG**
- **3 SnapEsi POS are likely resolved/treated infection**

SnapEsi™ Syphilis T/NT (Treponemal / Nontreponemal)

- Preliminary quantitative anti-cardiolipin results


Anti-cardiolipin assay in same array as anti-*T. pall* assay

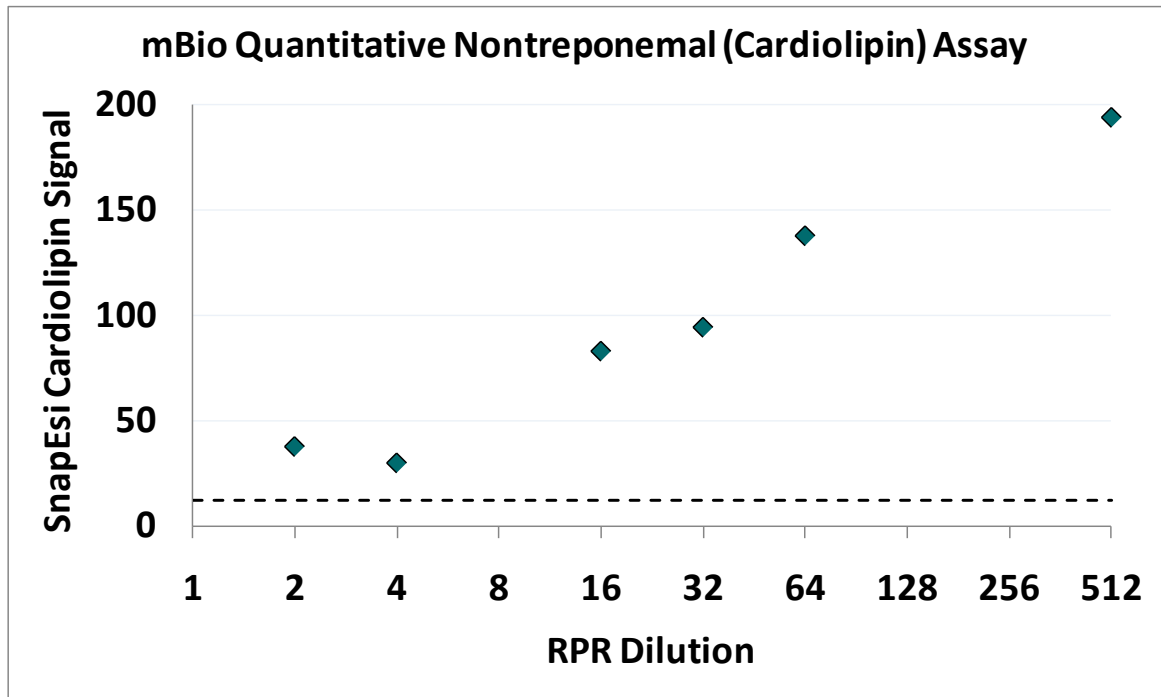

Dashed line is signal for pooled normal human serum.

mBio Development Pipeline: Quantitative Antigen Detection & Panel Expansion

- Hepatitis B Surface Antigen (HBsAg)
- **HIV-1 p24 direct antigen detection**
- HIV-2 serology
- Hepatitis C serology

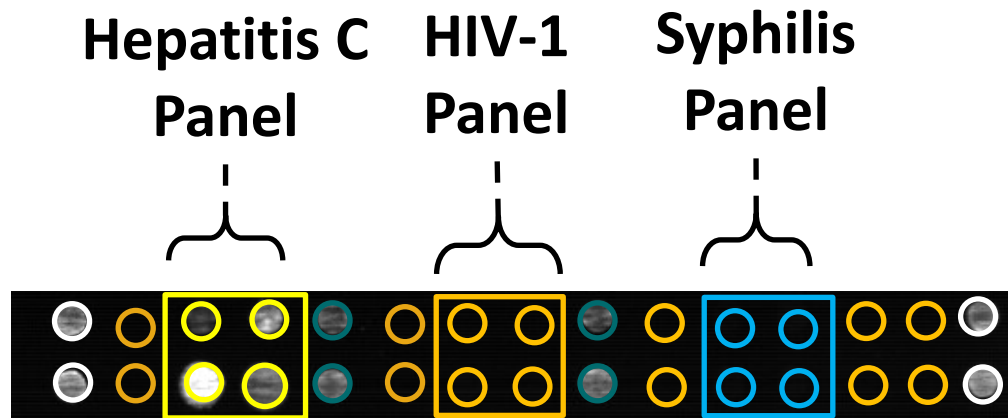



Summary

- **Simple, low-cost fluorescence imaging system with disposable assay cartridge**
- **mBio focus: HIV, coinfection, global health apps**

- **MAJOR THEMES**

- **Multiplexed assay panels at point of care**
- **Rapid test workflow & cost with lab quality output**



Acknowledgements - Collaborators

Clinical Collaborators:

Dr. Robert Schooley (UCSD)
Dr. Constance Benson (UCSD)
Dr. Sharon Reed (UCSD)

- **Dr. Emilia Noormahomed (Mozambique)**
- **Dr. Roberto Badaro (Brazil)**



PATH (Seattle) Center for Point-of-Care Diagnostics for Global Health

- **Matt Steele, Cori Barfield**

Planar Waveguide Technology

- **Prof. Jim Herron (University of Utah)**
- **Dr. Frances Ligler (Naval Research Lab)**



Acknowledgements

Funding

- NIH NIAID 2R44AI68543
- NIST 70NANB7H7053
- NIH NIAID U54AI065357
- PATH / NIH GHDx Center
- Precision Photonics Corp

Mike Lochhead, VP mBio

mike.lochhead@mbiodx.com

www.mbiodx.com



- Marie Delaney
- Jeff Ives
- Chad Greef
- Kevin Moll
- **Chris Myatt, CEO**
- Keagan Rowley
- Katie Todorof
- Kurt Vogel